

## **Tripod Instructions**

- 1. Remove tripod from carry bag.
- Open tripod by spreading legs and pushing down center crossbars. (Fig 1. & Fig 2.)
- Loosen Lever 1 just below red button & raise handle until top is parallel with floor (Fig 3 & Fig 4)
- 4. Tighten Lever 1.





Push down

Figure 1

Figure 2





## **Tripod Instructions**

- If center hook at base is touching the floor (Note: hook may have been removed), unlock lower lever at front leg and extend front leg <sup>1</sup>/<sub>2</sub>". (Fig 5)
- 6. Loosen Lever 2 at top of tripod legs & carefully turn small crank handle clockwise, raising tripod head approx. 3". This height will be adjusted during screening depending on the age of students being screened. (Fig 6) Do not use excessive force turning the crank handle.
- 7. Tighten Lever 2.
- Remove tripod platform top from rolling crate and take out black zippered case. There are several styles of platforms tops. Example (Fig 7)

Lever 2

Crank Handle

Figure 5



Center Hook

Front Leg





Figure 6

Figure 7




## **Tripod Instructions**


- Identify rectangular tripod mounting plate attached to bottom of platform top. (Fig 8)
- Look at top of tripod: Identify red release button, locking Lever 3, recessed track for sliding in tripod mounting plate and tapered track entry at rear of track. (Figs 9A & 9B)
- 11. Mount platform top onto tripod by sliding tripod mounting plate on bottom of platform into track on top of tripod from the rear until you hear a click. (Fig 10)
- 12. The platform top can move back and forth 2.75 inches. Center top and lock Lever 3.
- To remove, loosen Lever 3, push red button and slide top off of tripod.

#### Slide platform top onto tripod top



Figure 10

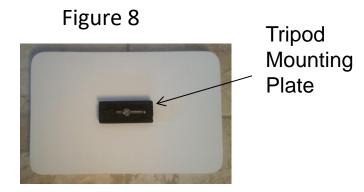

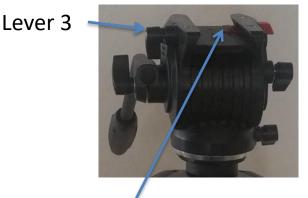

**Recessed Track** 

Red Button



Tapered Entry at Rear


Figures 9A & 9B

Locking Lever 3



### **Tripod Instructions**

14. To rotate the tripod left or right , loosen Lever 4, rotate tripod as needed and then tighten Lever 4. (See Figures 11 & 12)

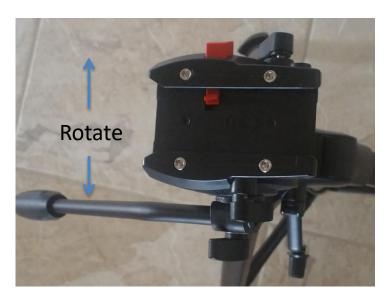



Lever 4 <sup>4</sup>

Figure 11

Figure 12